



Product Data Sheet

HX600 / HX601

Rev 1
 Effective date: 16.05.14
 Page 1 of 1
 Authorisation: RDP181

Product Description

The STF HexaLumen Automatic Endoscope Washer Disinfectant Test Kit (HX600) is a surrogate endoscope test device containing to be used in conjunction with HexaLumen Test Kit Indicators (HX601) used to verify the cleaning performance of automated endoscope washer disinfectors.

The device consists of six PTFE lumens that are of 1500 mm length; four having an ID (internal diameter) of 2 mm (blue) and two having an ID of 1 mm (red). At one end of the lumens, capsules are attached in which the indicators are placed. At the other end of the lumens is a luer lock fitting to be connected to the endoscope washer disinfectant.

Capsules for channel blockage tests are provided that are coloured red having a 0.5 mm hole. A hypodermic needle no longer than 25mm can be placed inside the capsule to restrict the diameter even further.



Specification

The HX601 test soil is equivalent in performance to the soils listed in ISO/TS 15883-5 Annex R and CFPP 01-06.

Applicable standards

ISO / TS 15883-5 Washer-Disinfectors: Test Soils And Methods For Demonstrating Cleaning Efficacy
CFPP 01-06 Choice Framework For Local Policy And Procedures – Decontamination Of Flexible Endoscopes: Testing Methods

Packaging HX600

Primary case contents:	1 HexaLumen Device	Size:	263mm x 263mm x 96mm	Weight	516g Gross
------------------------	--------------------	-------	----------------------	--------	------------

Packaging HX601

Primary case contents:	60 indicators	Size:	140mm x 90mm x 25mm	Weight	54g Gross
------------------------	---------------	-------	---------------------	--------	-----------

Recommended Storage

0 - 30 C 30 – 60 % RH (HX601 indicators only)

Product Expiry

Refer to product labelling for the expiry date.

Active Components

The dried test soil formula contains protein, lipid and polysaccharide. None of the substances used in the formulation are known carcinogens, nor do they contain any heavy metals. If contact is made with the indicating ink, wash the skin with soap and water. If ingested wash out the mouth thoroughly and give plenty of water to drink.

Safe Usage

The STF HexaLumen Automatic Endoscope Washer Disinfectant Test Kit Indicator contains inert ingredients, colouring dyes, egg proteinaceous and lipid materials. As such it does not present a risk to health when used as directed. Avoid contact with skin and eyes.

Do not ingest. If material inadvertently gets into eyes flush with water and seek medical advice.

The product and packaging do not contain natural rubber latex (NRL) or dry natural rubber (DNR).

Contains no blood products. Contains no porcine products.

Disposal

Dispose of used/expired indicators using normal waste disposal.