

Technical Data Sheet

medi ROM Kidz



medi GmbH & Co. KG
Medicusstrasse 1
95448 Bayreuth
Germany
www.medi.de

Product Management
phone +49 921 912 -
fax +49 921 912 -

Rev: 11/2022

1 Product description

Intended Purpose

medi ROM Kidz is a knee brace for mobilisation.

Indications

All indications which require early functional mobilisation and limitation of the range of motion of the knee joint, such as:

- Following ligament injury (post-traumatic / post-operative)
- Following meniscsal injury (post-traumatic / post-operative)
- Following tendon injury (post-traumatic / post-operative)

Contraindications

None known at present

Risks / Side Effects:

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paresthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localized skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Soap residues, lotions and ointments can cause skin irritation and material wear.

1.1 Material composition

Aluminium, PU-foam, Cotton

1.2 Presentations

Color: blue

short G.180.0K1

long G.180.0K2

Length of orthosis	short (25 cm)	long (35 cm)
Weight	333 g	379 g
Strap 1	49 cm	49 cm
Strap 2	46 cm	46 cm
Strap 3	43 cm	43 cm
Strap 4	40 cm	40 cm
Pad width bottom	44 cm	44 cm
Pad width top	58 cm	58 cm
Length	33 cm	38 cm

1.3 Features

Open wrap design for easy don and doff - without to flex the knee

Individually adjustable to leg circumference by three part design and hook-and-loop fastener

Comfortable, skin friendly and washable material

Removable and adjustable stabilisation splints

Modern and child-friendly design.



Especially designed to meet children's needs

2 Safety instructions

Important notes:

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds and use it only as instructed by your doctor or orthotist.

3 Storing conditions

Keep the product in a dry place and do not expose to direct sunlight.  

4 Chemical substances

Contact with chemicals (e.g. benzole, methanol, salicylic acid in high concentrations) may have adverse actions on the effect of the product. Do not apply ointments or oils before putting on the product.

5 Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the paddings by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.



6 Packaging/Labeling

Product is sent out in plastic bag with CE mark und manufacturer address. Product and plastic bag labeled for clear identification and traceability.
Fitting instructions included.

7 Shelf Life

Expiration date for storage: 60 months

8 Disposal

The product can be disposed of in the domestic waste.



Please ensure a proper and suitable disposal of the used products to prevent any risk to human beings and the environment.