

Technical Data Sheet

Omomed

Rev: 11/2022

medi GmbH & Co. KG
 Medicusstrasse 1
 95448 Bayreuth
 Germany
www.medi.de



Product Management
 phone +49 921 912 -
 fax +49 921 912 -

1 Product description

Intended purpose

Omomed® is a shoulder joint compression support with additional functional elements.

Indications

All indications in which at least one direction of movement of the shoulder is restricted, secured or changed, such as:

- Nonsurgical with anterior shoulder dislocation and chronic anterior subluxation
- Painful omarthrosis
- Postoperative after shoulder stabilisations

Contraindications

Nonsurgical and postoperative treatment of posterior shoulder luxation

Risks / Side Effects:

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paresthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localized skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Soap residues, lotions and ointments can cause skin irritation and material wear.

1.1 Material composition

Polyamid, polyester, elastane

1.2 Presentations

size	article number left	article number right	circumference upper arm
0	K.302.200	K.303.200	up to 26 cm
I	K.302.201	K.303.201	26 – 29 cm
II	K.302.202	K.303.202	29 – 33 cm
III	K.302.203	K.303.203	33 – 36 cm
IV	K.302.204	K.303.204	36 – 39 cm

1.3 Product measurements

length straps incl. hook and loop

size	length along shoulder	upper arm	length strap white	length strap waist long	length strap waist short	length strap behind
0	48cm	10cm	44cm	98cm	32cm	52cm
I	49cm	11cm	45cm	99cm	32cm	52cm
II	52cm	12cm	45cm	99cm	32cm	52cm
III	54cm	12,3cm	55cm	100cm	32cm	53cm
IV	56cm	14,5cm	55cm	100cm	32cm	53cm
size	length along shoulder	upper arm	length strap white	length strap waist long	length strap waist short	length strap behind
0	48cm	10cm	44cm	98cm	32cm	52cm
I	49cm	11cm	45cm	99cm	32cm	52cm
II	52cm	12cm	45cm	99cm	32cm	52cm
III	54cm	12,3cm	55cm	100cm	32cm	53cm
IV	56cm	14,5cm	55cm	100cm	32cm	53cm

1.4 Features

- Flexible strap system permits individual adjustment and variable limitation of movement at the shoulder joint
- Intelligent strap system ensures comfortable wearing, especially in the armpit and chest area
- Can easily be worn under clothing
- Clima Comfort*:



- Clima Comfort ensures that the garment is comfortable to wear. Modern fibre technology wicks moisture away from the skin's surface quickly and ensures a high level of breathability.

*Scientifically tested by French institute IFTH.

- Clima Fresh:
 - The integrated Clima Fresh function has an odour-inhibiting effect. This is achieved via special thread technology or via special processing during the production process. These have an antibacterial effect and thus inhibit odour-forming bacteria within the fabric.

2 Safety instructions



Important notes

This medical device  is made for single patient use only . If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds and use it only as instructed by your doctor or orthotist.

Note on wearing

It is suitable for continuous use over a number of weeks. But during longer periods of rest (e.g. when sleeping) the support should be removed.

3 Storing conditions

Keep the product in a cool, dry place and do not expose to direct sunlight.  

4 Chemical substances

Contact with chemicals (e. g. benzole, methanol, salicylic acid in high concentrations) may have adverse actions on the effect of the product. Do not apply ointments or oils before putting on the product.

5 Care instructions

Velcro fasteners should be closed for washing. We recommend using a laundry bag. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean



6 Packaging/Labeling

Product is sent out in folded box with CE mark and manufacturer address.

Product and box labeled for clear identification and traceability.

Fitting instruction included.

7 Shelf Life

Expiration date for storage: 42 months

8 Disposal

The product can be disposed of in the domestic waste.

