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Types: Sterile and Non-sterile Leg Bags, Bed Bags, Closed-System Bags

DESCRIPTION

The Urine Bag is attached to the end of a urinary catheter and acts as a reservoir for the temporary collection of urine. The product is used in hospitals or in home healthcare with patients using bladder- or male external catheters (sheaths).

FEATURES

- Can be used in hospital and home healthcare settings.
- Constitutes an efficient solution for transient or chronic urinary incontinence.
- Leg Bags can be comfortably secured to the leg and preserve the user's need for freedom and discretion. Available in diverse options, in sterile and non-sterile versions.
- Sterile Closed-System Bags are designed to accommodate the hospital needs with larger volumes, needle-free sampling port, options with integrated hanger and pump for bladder irrigation.

INTENDED USE

The Urine Bag is used for the collection and temporary storage of urine. It is intended to be used with an indwelling catheter in incontinent patients, cannot urinate normally, or are requiring continuous bladder drainage.

INDICATIONS

The Urine Bag is connected to a bladder catheter or male external catheter for collection and measurement of the urine output.

CLINICAL BENEFITS

The Urine Bag is suitable for patients suffering from chronic pathologies or conditions that impair the capability to use the toilet. The clinical benefit associated with the device is defined as direct: it is intended to maintain a person's mobility, dignity, and comfort.

PATIENT TARGET GROUP

Neonate, infant, child, adult patients.

INTENDED USER

The Urine Bag must be used by qualified professionals or adequately trained non-professional caregivers.

CONTRAINDICATIONS

There are no known contraindications.

ADVERSE REACTIONS

No adverse reactions have been reported.

WARNINGS / PRECAUTIONS

- Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury.
- Do not use if the sterile packaging is damaged or unintentionally opened before use.
- Do not use if the product is damaged or irregularly shaped.
- Do not use after the expiry date.
- For single patient use only.
- To promote smooth drainage, make sure that the urine bag is secured below bladder level and that the tube is not kinked or crushed.
- For proper use of the product, follow hospital procedures or consult healthcare personnel.
- Do not reuse, reprocess, or resterilise. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in injury, illness or death of the patient. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.
- Dispose of product and packaging in accordance with hospital administrative and/or

local government policy. Empty the bag before disposal.

- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent authority of the user's place.

DIRECTIONS FOR USE

- Open the package and take out the bag.
- Remove the protective cap from the tube, wipe the connector with disinfectant and firmly insert it into the drainage funnel of the urinary catheter.

Leg Bag

- Fasten the bag to the thigh or the calf in a comfortable position using the elastic straps. Ensure that the bag is below the bladder level and the tube does not pull or kink.
- Empty the bag at regular intervals when it is a third to half full. If required, record the amount of collected urine.
- To empty the bag, hold it over the toilet or a suitable hygienic disposal. Open the bottom outlet and let all the urine flow out. Tightly close the bottom outlet and wipe it with disinfectant.
- Replace the leg bag on a schedule (e.g., weekly), and as needed (e.g., appears soiled or damaged).

Bed Bag / Closed-System Bag

- Hang the bag on the bed rail using a suitable hook or secure it on a bag stand. Ensure that the bag is below the bladder level and the tube does not pull, kink or form loops.
- Empty the bag according to hospital protocol

or at regular intervals when it is a third to half full. If required, record the amount of collected urine.

- To empty the bag, position a suitable hygienic disposal under it and open the bottom outlet to let all the urine flow out. Tightly close the bottom outlet and wipe it with disinfectant.
- Replace the bag according to hospital protocol. For bags used at home follow local home healthcare indications.

Exchange of Leg Bag, Bed Bag, Close-System Bag

- If available, close the tube clamp and disconnect the bag from the catheter.
- Wipe the drainage funnel of the catheter with disinfectant.
- Connect a new bag to the catheter following the instructions above.
- Dispose of the previous bag and its content as appropriate.

STORAGE CONDITIONS

- Store in a dark, cool and dry place, keep away from sunlight.
- Protect product from moisture and excessive heat.
- Avoid prolonged exposure to ultraviolet, sunlight and fluorescent light.
- Store in manner preventing crushing.

KEY PERFORMANCE CHARACTERISTICS

Leak test under impact: the urine bag is filled with water and dropped onto a smooth, rigid surface. The bag must not leak.

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UNOQUIP Pty Ltd
Filial af UNOQUIP GmbH, Switzerland
Well Lead Medical Co.,Ltd.
QS Engineering AG
Corp.GmbH(Europe)
SUNGO Certification Company Limited